



# FIVALI SPRAINED WRIST BRACE

## Instructions for Use

Please read the donning instructions and warnings carefully prior to use. To ensure that the brace works properly, follow all instructions.

**PLEASE MAKE SURE YOUR HANDS ARE CLEAN AND DRY FOR APPLICATION**

**1**



Pull the button upwards and release all the cords;

**2**



Place your hand through the wrist brace while placing your thumb through the A as shown, adjusting to the proper position;

**3**




Press the button to lock it in place and you will hear a "Click";

**4**



Turn the knob clockwise to wrap the wrist splint tightly around your wrist;

**5**

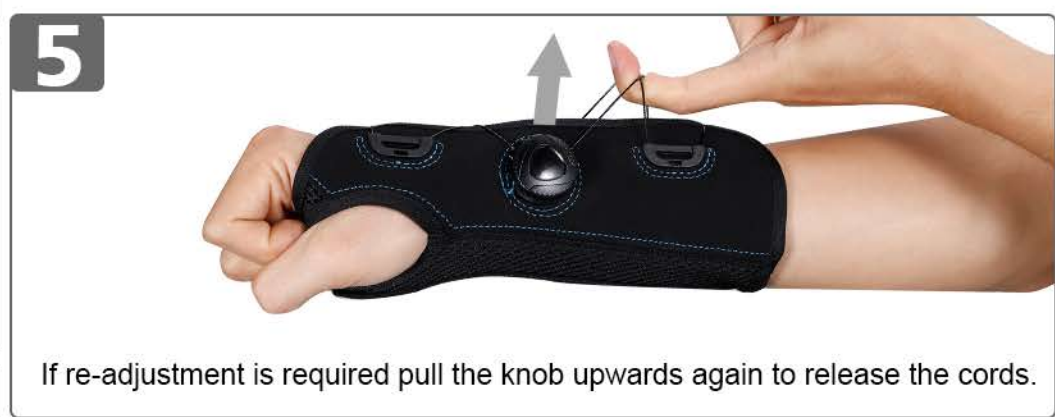

If re-adjustment is required pull the knob upwards again to release the cords.

### CARE & WASHING INSTRUCTIONS

- Make sure the brace fits snugly but comfortably around your wrist.
- Take breaks from wearing the brace throughout the day to allow your skin to breathe and prevent any potential discomfort.
- Clean your wrist brace regularly to prevent the buildup of sweat, dirt, or bacteria that can cause odors or skin irritation.
- If you have a pre-existing wrist condition or injury, consult with your doctor before purchasing and using a wrist brace to ensure it is appropriate for your specific needs.



### WARNINGS ⚠

- Neoprene supports should not be worn by individuals prone to dermatitis or allergic to neoprene products.
- If you experience pain, swelling, skin irritation or any other adverse reaction while using this product, consult your healthcare professional immediately.

### SINGLE USER ONLY